Read all instructions prior to collection.

This product is designed for the collection of human DNA from saliva samples.

Contents: Kit contains stabilizing liquid.

Warnings and precautions: Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at www.dnagenotek.com.

Storage: 15°C / 25°C

Summary and explanation of the kit:

Oragene-DISCOVER is a self-collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Collection precautions:

Do NOT eat, drink, smoke or chew gum for 30 minutes before giving your saliva sample.

Do NOT remove the plastic film from the cap.

Procedure:





Most people take between 2 and 5 minutes to deliver a saliva sample following steps 1 to 4.

1 Start giving your saliva sample.

2 Spit until the amount of liquid saliva (not bubbles) reaches the level shown in picture #2.

3 Put the container on a flat surface. Screw the cap onto the container. Make sure that the cap is closed tightly, as shown in picture #3c. If you have difficulty closing the cap completely, turn cap slightly counter-clockwise then try again.

4 Shake gently for 10 seconds. Recycle outer packaging.

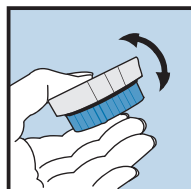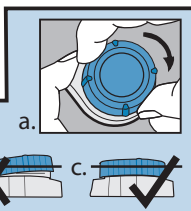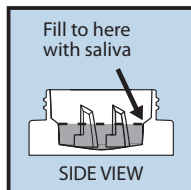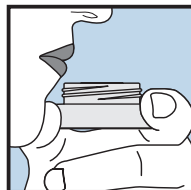DNA genotek

Made in Canada
DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2
Tel.: +1.613.723.5757
info@dnagenotek.com
www.dnagenotek.com

Patent (www.dnagenotek.com/legalnotices)

PD-PR-00154 Issue 6/2018-08

Label legend:

	Consult package insert
	Collect saliva by (Use by)
	Catalog number
	Storage instructions
	Manufacturer
	Lot number

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