

Summary and explanation of the kit:

OMNIGene-GUT provides the materials and instructions for collecting and stabilizing microbial DNA from a fecal sample.

Warnings and precautions:

- FOR EXTERNAL USE ONLY.
- Do NOT remove the yellow tube top from the tube.
- Do NOT spill the stabilizing liquid in the tube.
- Wash with water if liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at www.dnagenotek.com
- If collecting a liquid fecal sample, see separately provided user instructions.
- Small items may pose a choking hazard.

Storage: Room temperature storage pre- and post-collection 15°C to 25°C (60°F to 78°F).

Ship in accordance to applicable regulations covering transport of biological specimens.

Label legend:

- Collect sample by (Use by)
- Catalog number
- Manufacturer
- Caution, consult instructions for use
- Lot number
- Authorized Representative
- 15°C / 25°C Storage instructions
- 60°F / 78°F Storage instructions

USER INSTRUCTIONS

Read all instructions prior to collection

Other languages are available at www.dnagenotek.com


Procedure:




- 1 IMPORTANT PREPARATIONS:**
- Empty your bladder before beginning the collection.
 - Collect fecal sample free of urine or toilet water.
 - Utilize any toilet accessory which may be supplied with the collection device.
 - Toilet paper or tissues may be required.



- 2** While holding the yellow tube top, unscrew ONLY the purple cap and set aside for later use.
- IMPORTANT:** Do NOT remove the yellow tube top. Do NOT spill the stabilizing liquid in the tube.

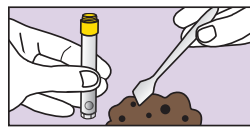

- 3** Use the spatula to collect a small amount of fecal sample.
-
- Actual size of fecal sample.

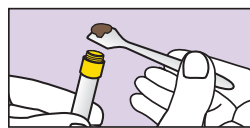


- 4** Transfer the fecal sample into the top portion of the yellow tube top. Repeat until the sample fills the yellow tube top.
- IMPORTANT:** Do NOT push sample into the tube.




- 5** Scrape horizontally across the tube top to level the sample and remove any excess.
- Wipe exterior of tube and top with toilet paper or tissue as needed.




- 6** Pick up the purple cap and screw onto the yellow tube top until tightly closed.

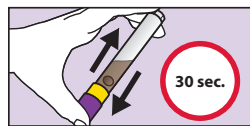

- 7** For a **minimum of 30 seconds**, shake the sealed tube as hard and fast as possible in a back and forth motion.

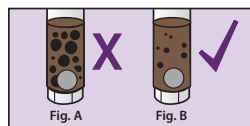

- 8** The fecal sample will be mixed with the stabilizing liquid in the tube; not all particles will dissolve.
- IMPORTANT:** Continue shaking if large particles remain as shown in Figure A.

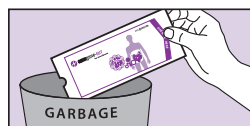

- 9** Place spatula in original packaging or wrap in toilet paper and discard in garbage.
- IMPORTANT:** Send the sample for processing following the delivery instructions supplied separately by the kit provider.