
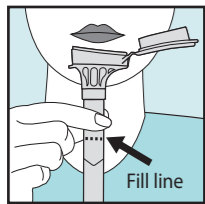
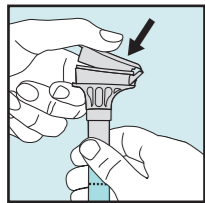
1


2

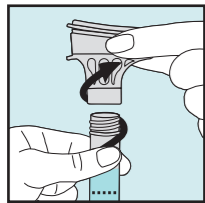
3


4

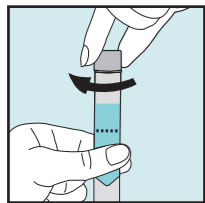5

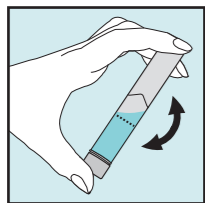ENGLISH
User instructions

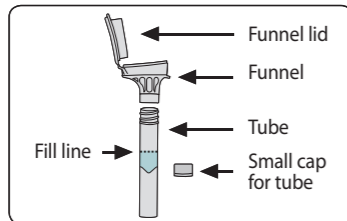Read all instructions prior to collection.

Intended use: For the collection of human DNA from saliva samples.

Contents: Kit contains stabilizing liquid.

Warnings and precautions: Rx only. Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at www.dnagenotek.com.

Small cap may pose a choking hazard.

Storage: 15°C / 30°C

Summary and explanation of the kit:

Oragene-Dx is a self-collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Collection precautions:

Do NOT eat, drink, smoke or chew gum for 30 minutes before giving your saliva sample.

Do NOT remove the plastic film from the funnel lid.

Procedure:

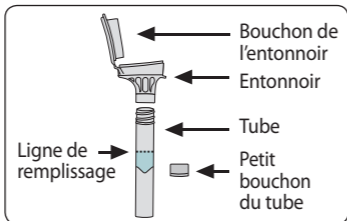
Most people take between 2 and 5 minutes to deliver a saliva sample following steps 1 to 5.

- Spit into funnel until the amount of liquid saliva (not bubbles) reaches the fill line shown in picture #1.
- Hold the tube upright with one hand. Close the funnel lid with the other hand (as shown) by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva. Make sure that the lid is closed tightly.
- Hold the tube upright. Unscrew the funnel from the tube.
- Use the small cap to close the tube tightly.
- Shake the capped tube for 5 seconds. Discard the funnel.

Label legend:

- Consult package insert
- Collect saliva by (Use by)
- In vitro diagnostic medical device
- Catalog number
- CE Marking
- Caution, consult instructions for use
- Storage instructions
- Authorized Representative
- Manufacturer
- Lot number

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