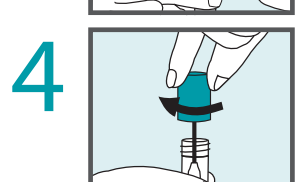
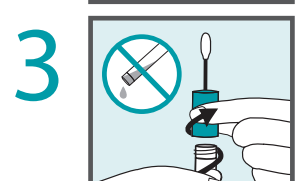
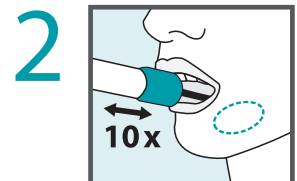
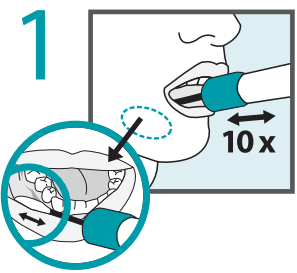
ORACollect[®]-Dx

For In Vitro Diagnostic Use
For Prescription and
Direct-to-Consumer Use

DNAGENOTEK[™]

REF: OCD-100A
DNA Genetec Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Tel: +1.613.723.5757
info@dnagenotek.com
www.dnagenotek.com

Patent (www.dnagenotek.com/legalnotices)
PD-PR-01123 Issue 3/2023-01

ENGLISH User instructions

Read all instructions prior to collection.

Collection precautions:
Donor should NOT eat, drink, smoke, or chew gum for 30 minutes before collecting saliva sample.

Ensure the sponge tip does NOT come into contact with any surface prior to collection.

Intended use: ORACollect[®]-Dx is intended for the collection of saliva samples for diagnostic testing of human DNA. Saliva samples may be collected by a healthcare professional or non-healthcare professional, such as a lay user. Saliva samples collected using ORACollect[®]-Dx are stabilized and isolated for use in downstream diagnostic testing applications. Saliva samples collected using ORACollect[®]-Dx can be transported and/or stored at ambient conditions.

Contents: 1 kit containing stabilizing liquid.

Warnings and precautions:

- Read all instructions carefully prior to sample collection; deviation may result in inadequate sample and impact DNA yield.
- For use in individuals 18 years of age and older.
- Choking hazard. Caution should be used when inserting sponge into the mouth.
- Donors with xerostomia (dry mouth) may not collect adequate sample using these instructions, resulting in lower DNA yield and an invalid sample for use.
- Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at www.dnagenotek.com.

Report any serious incident to DNA Genetec and the competent authority in your country.

Storage: 15°C-25°C (59°F-77°F)

Procedure for saliva sample collection:

- Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close-up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.
- Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.
- Hold the tube upright to prevent the stabilizing liquid inside the tube from spilling. Unscrew the blue cap from the collection tube without touching the sponge.
- Turn the cap upside down, insert the sponge into the tube and close cap tightly.
- Invert the capped tube and shake vigorously 10 times.

Label legend:

REF	Catalog number
REF	In vitro diagnostic medical device*
CE	CE marking
UKCA	UKCA marking
MD	Manufacturer
MD	Medical device*
⊗	Do not re-use
IF	Consult instructions for use
IF	Collect sample by (Use by)
LOT	Lot number
REF	Authorized Representative
REF	Swiss Authorized Representative
REF	Unique Device Identifier

*Per applicable regulatory requirements, this device may be a medical device or in vitro diagnostic medical device.

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OraSure Technologies, Inc., all rights reserved.
Limited c/o Cr360 – UL International, Compass House,
Arazy Group Swiss GmbH
© 2023 DNA Genotek Inc., a subsidiary of