

**Contents:** 1 kit containing stabilizing liquid.

**Summary and explanation of the kit:**

OMNigene-GUT DNA and RNA provides the materials and instructions for collecting and stabilizing microbial DNA and RNA from a fecal sample.







**Warnings and precautions:**

- FOR EXTERNAL USE ONLY.
- Do NOT remove the yellow tube top from the tube.
- Do NOT spill the stabilizing liquid in the tube.
- Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at [www.dnagenotek.com](http://www.dnagenotek.com).
- If collecting a liquid fecal sample, see separately provided user instructions.
- Small items may pose a choking hazard.

**Storage:** Room temperature storage pre- and post-collection 15°C ± 25°C (59°F ± 77°F).

Ship in accordance with applicable regulations covering transport of biological specimens.

**Label legend:**

-  Collect sample by (Use by)
-  Catalog number
-  Lot number
-  Manufacturer
-  Caution, consult instructions for use
-  Consult package insert

15°C ± 25°C Storage instructions  
(59°F ± 77°F)

**USER INSTRUCTIONS**

**Read all instructions prior to collection.**

**Proceed to transferring the fecal sample as soon as possible after defecation.**

**Procedure:**

- 

**1 IMPORTANT PREPARATIONS:**

  - Empty your bladder before beginning the collection.
  - Collect fecal sample free of urine or toilet water.
  - Utilize any toilet accessory that may be supplied with the collection device.
  - Toilet paper or tissues may be required.
- 

**2** While holding the yellow tube top, unscrew ONLY the purple cap and set aside for later use.

**IMPORTANT:** Do NOT remove the yellow tube top. Tube is not designed to stand upright. Do NOT spill the stabilizing liquid in the tube.


- 

**3** Use the spatula to collect a small amount of fecal sample.



Actual size of fecal sample.
- 

**4** Transfer the fecal sample into the top portion of the yellow tube top. Repeat until the sample fills the yellow tube top.

**IMPORTANT:** Do NOT push sample into the tube.


- 

**5** Scrape horizontally across the tube top to level the sample and remove any excess. Wipe exterior of tube and top with toilet paper or tissue as needed.
- 

**6** Pick up the purple cap and screw onto the yellow tube top until tightly closed.


- 

**7** For a **minimum of 30 seconds**, shake the sealed tube as hard and fast as possible in a back and forth motion.
- 

**8** Place spatula in original packaging or wrap in toilet paper and discard in garbage.

**IMPORTANT:** Send the sample for processing following the delivery instructions supplied separately by the kit provider.