English – Procedure for saliva sample collection	2
Français – Procédure de prélèvement d'un échantillon de salive	3
Español – Procedimiento para la recogida de la muestra de saliva	4
Deutsch – Verfahren zur Entnahme einer Speichelprobe.....	5
Norsk – Prosedyre for innsamling av spyttprøve.....	6
Svenska – Procedur för insamling av salivprov.....	7
Dansk – Procedure til indsamling af spyttprøve	8
Nederlands – Procedure voor afname van speekselmonster.....	9
Česky – Postup odběru vzorku slin.....	10
Italiano – Procedura per la raccolta del campione di saliva	11
Polski – Procedura pobierania próbek śliny	12
Português – Procedimento para a recolha de amostras de saliva.....	13
Português – Brasil – Procedimento para a recolha de amostras de saliva	14
العربية – إجراء جمع عينة اللعاب	15
中文 – 唾液样本采集步骤.....	16
한글 – 타액 샘플 채취용 절차	17
日本語 – 唾液のサンプル採取方法	18
Български – Процедура за вземане на проба от слюнка.....	19
Hrvatski – Postupak uzimanja uzorka pljuvačke.....	20
Eesti keel – Süljeproovi võtmise protseduur	21
Suomi – Sylkinäytteen ottaminen.....	22
Ελληνικά – Διαδικασία λήψης δείγματος σιέλου.....	23
Magyar – Nyálmintagyűjtésének eljárása	24
Gaeilge – Gnáthamh maidir le bailiú samplaí seile	25
Latviešu valoda – Procedūra siekalu parauga paņemšanai.....	26
Lietuvių k. – Seilių mėginio paėmimo procedūra.....	27
Malti – Procedura għil-gbir tal-bżieq.....	28
Română – Procedura de colectare a probelor de salivă.....	29
РУССКИЙ – Процедура сбора образца слюны.....	30
Slovensky – Postup na odber vzorky slín.....	31
Slovenščina – Postopek za jemanje vzorca sline.....	32
עברית – הליך לאיסוף דגימת רוק	33

Collection precautions:

Donor should NOT eat, drink, smoke, or chew gum for 30 minutes before collecting saliva sample.

Ensure the sponge tip does NOT come into contact with any surface prior to collection.

Intended use: ORAc collect™ Dx is intended for the collection of saliva samples for diagnostic testing of human DNA. Saliva samples may be collected by a healthcare professional or non-healthcare professional, such as a lay user. Saliva samples collected using ORAc collect™ Dx are stabilized and isolated for use in downstream diagnostic testing applications. Saliva samples collected using ORAc collect™ Dx can be transported and/or stored at ambient conditions.

Contents: 1 kit containing 1 mL stabilizing liquid.

Warnings and precautions:

- Read all instructions carefully prior to sample collection; deviation may result in inadequate sample and impact DNA yield.
- For use in individuals 18 years of age and older.
- Choking hazard. Caution should be used when inserting sponge into the mouth.
- Donors with xerostomia (dry mouth) may not collect adequate sample using these instructions, resulting in lower DNA yield and an invalid sample for use.
- Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at www.dnagenotek.com.

Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15°C-25°C (59°F-77°F)

Label legend:

	Catalog number
	In vitro diagnostic medical device**
	CE marking
	UKCA marking
	Manufacturer
	Medical device*
	Do not re-use
	Consult instructions for use
	Collect sample by (Use by)
	Lot number
	Authorized Representative
	Swiss Authorized Representative
	Unique Device Identifier

*Per applicable regulatory requirements, the device contains a swab component, which is a *medical device, and a collection tube component, which is an **in vitro device.*

ENGLISH

Procedure for saliva sample collection:

<p>1 10 x</p>	<p>Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close-up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.</p>
<p>2 10 x</p>	<p>Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.</p>
<p>3</p>	<p>Hold the tube upright to prevent the stabilizing liquid inside the tube from spilling. Unscrew the blue cap from the collection tube without touching the sponge.</p>
<p>4</p>	<p>Turn the cap upside down, insert the sponge into the tube and close cap tightly.</p>
<p>5 10 x</p>	<p>Invert the capped tube and shake vigorously 10 times.</p>

For In Vitro Diagnostic Use
For Prescription and Direct-to-Consumer Use

DNAGENOTEK™

Superior samples
Proven performance

■ DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com

Novosanis NV, Bijkhoevelaan 32c, 2110 Wijnegem, Belgium Email: EUAR@novosanis.com

UK Responsible Person: Emergo Consulting (UK) Limited c/o Cr360 – UL International, Compass House, Vision Park Histon, Cambridge, CB24 9BZ

Arazy Group Swiss GmbH, Bruderholzallee 53, 4059 Basel, Schweiz Email: swiss.ar@arazygroup.com

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

ORAc collect and DNA Genotek are trademarks of DNA Genotek Inc. Some DNA Genotek products may not be available in all geographic regions; contact your sales representative for details. All DNA Genotek protocols, white papers and application notes are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legalnotices)
© 2024 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.
PD-PR-506 Issue 9/2024-01

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