Version 2024-07A

Colli-Pee® FV-5004-RUO


REF

N00290_RUO


RUO

COLLI-PEE is a registered trademark of DNA Genotek Inc.

I.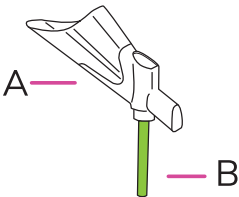

II.

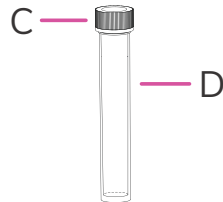Patent (www.dnagenotek.com/us/company/legal-notices.html)
© 2024 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

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EN
FIRST-VOID URINE SELF-SAMPLING DEVICE

For standardized, volumetric collection of first-void urine samples.
Read all instructions prior to collection.

ONE PACKAGE CONTAINS

I. Colli-Pee® funnel
A. Housing, B. Floater
II. Collector tube
C. Tube cap, D. Collector tube

CAUTION

Do NOT wash your intimate zone thoroughly before urinating.
Do NOT urinate 1 hour prior to use.
Do NOT use if the packaging is damaged.
For single use only.

User instructions

- Unscrew the tube cap (C).
- Screw the collector tube (D) on the Colli-Pee® funnel.
- Collect urine: The device will automatically collect the correct volume. There is no need to interrupt the urine stream. Excess urine that no longer fits into the collector tube, will flow into the toilet via the other end of the housing.
- Disconnect the collector tube from the Colli-Pee® funnel.
- Close the tube tightly. Tube cap (C) should be screwed on tightly.
- Dispose the Colli-Pee® funnel according to local regulations for plastic waste.
- Wash your hands.

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