
Colli-Pee® FV-5010

REF N00309


CE IVD UKRA

COLLI-PEE is a registered trademark of DNA Genotek Inc.

I.

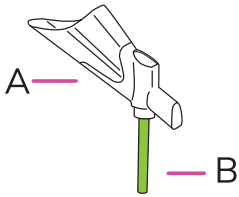II.







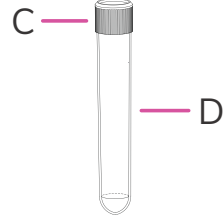
Patent (www.dnagenotek.com/us/company/legal-notices.html)
© 2024 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

EC REP Qarad EC-REP BV
Pas 257
2440 Geel
Belgium
info@qarad.com

UK RP Qarad UK Ltd.
8 Northumberland Ave
Westminster, London WC2N 5BY
United Kingdom
info@qarad.com

DNagenotek™
www.dnagenotek.com

 DNA Genotek Inc.
3000-500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

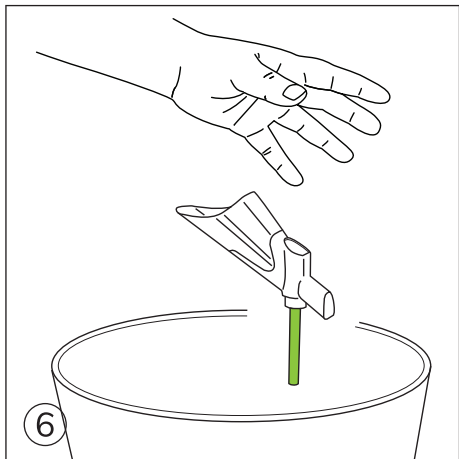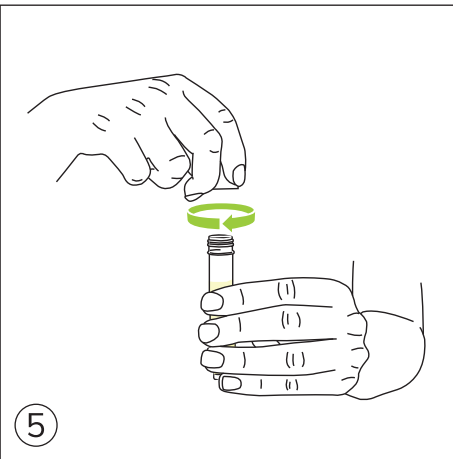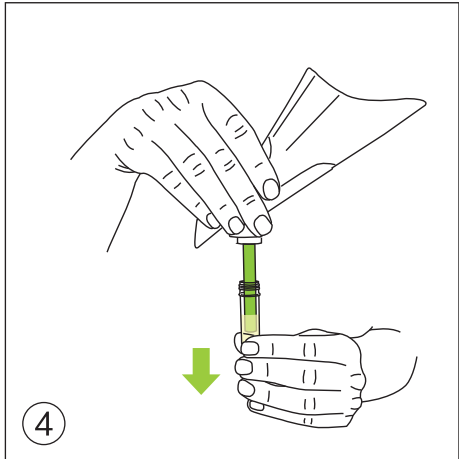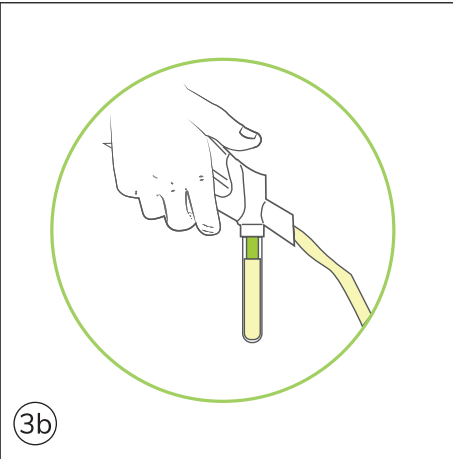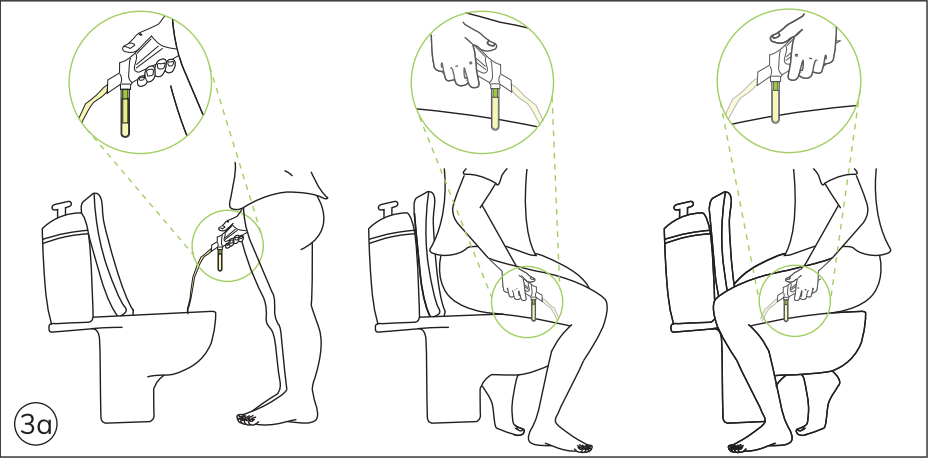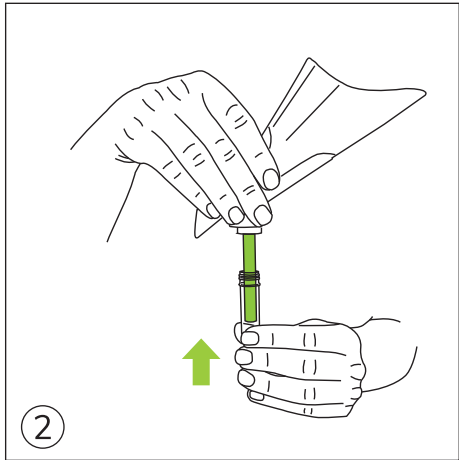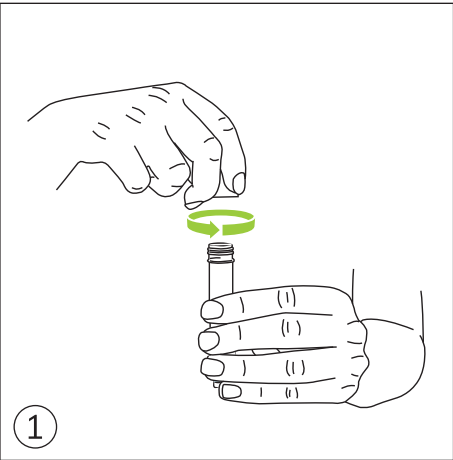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EN
FIRST-VOID URINE SELF-SAMPLING DEVICE

For standardized, volumetric collection of first-void urine samples.
Read all instructions prior to collection.

ONE PACKAGE CONTAINS

- I. Colli-Pee® funnel**
A. Housing, B. Floater
II. Collector tube
C. Tube cap, D. Collector tube

CAUTION

Do NOT wash your intimate zone thoroughly before urinating.
Do NOT urinate 1 hour prior to use.
Do NOT use if the packaging is damaged.
For single use only.

User instructions

- Unscrew the tube cap (C).
- Push the collector tube (D) on the Colli-Pee® funnel.
- Collect urine: The device will automatically collect the correct volume. There is no need to interrupt the urine stream. Excess urine that no longer fits into the collector tube, will flow into the toilet via the other end of the housing.*
- Disconnect the collector tube from the Colli-Pee® funnel.
- Close the tube tightly. Tube cap (C) should be screwed on tightly.
- Dispose the Colli-Pee® funnel according to local regulations for plastic waste.
- Wash your hands.

*In case of any serious incident with the device, please inform DNA Genotek and FAMHP: <https://www.famhp.be/en>

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