
Colli-Pee® FV-5010-RUO


REF N00309\_RUO

RUO

COLLI-PEE is a registered trademark of DNA Genotek Inc.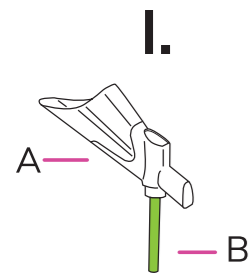

II.

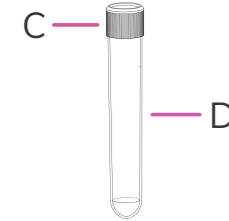Patent ([www.dnagenotek.com/us/company/legal-notice.html](http://www.dnagenotek.com/us/company/legal-notice.html))  
© 2024 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

EC REP





Qarad EC-REP BV  
Pas 257  
2440 Geel  
Belgium  
info@qarad.com

UK RP

Qarad UK Ltd.  
8 Northumberland Ave  
Westminster, London WC2N 5BY  
United Kingdom  
info@qarad.com

**DNAGENOTEK™**  
[www.dnagenotek.com](http://www.dnagenotek.com)

DNA Genotek Inc.  
3000-500 Palladium Drive  
Ottawa, ON, Canada K2V 1C2

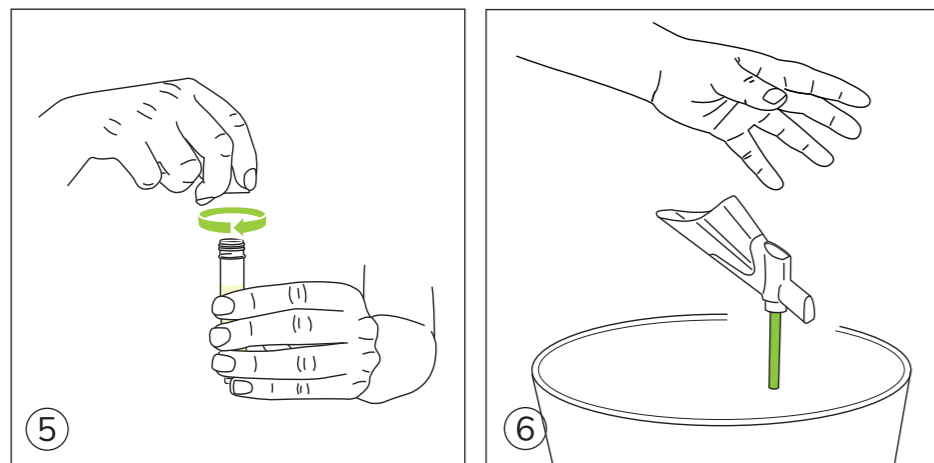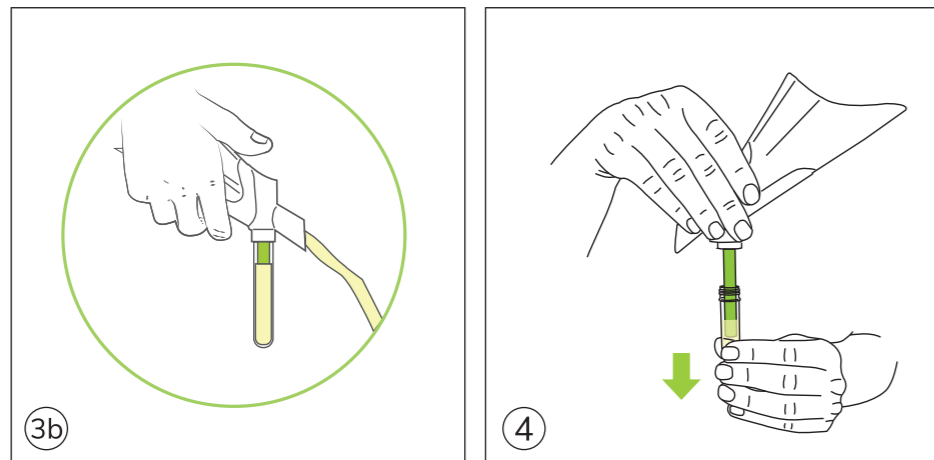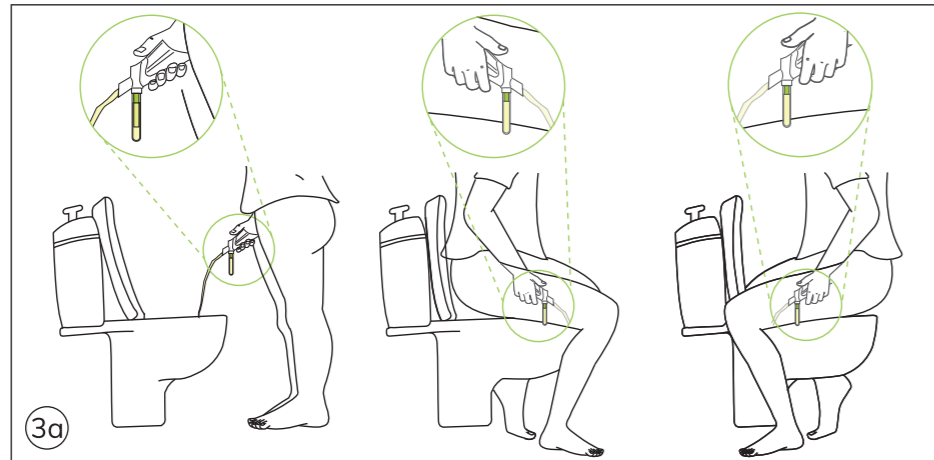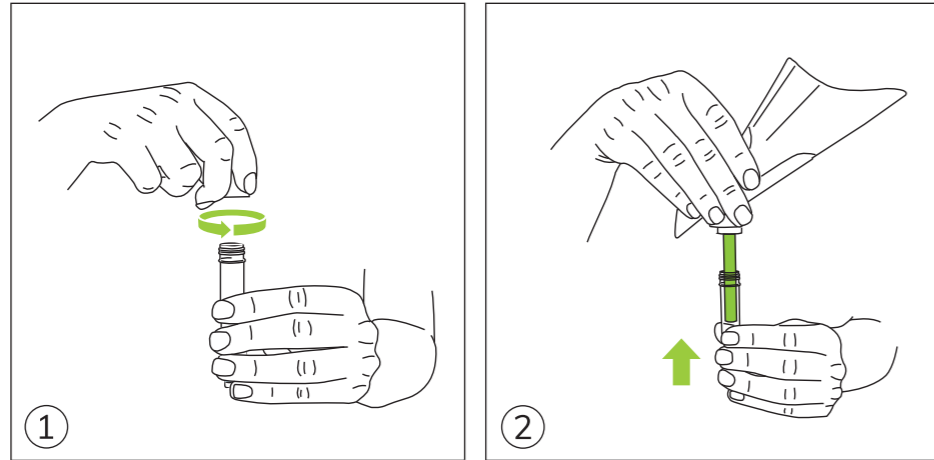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## EN FIRST-VOID URINE SELF-SAMPLING DEVICE

For standardized, volumetric collection of first-void urine samples.  
Read all instructions prior to collection.

### ONE PACKAGE CONTAINS

- I. Colli-Pee® funnel**  
A. Housing, B. Floater  
**II. Collector tube**  
C. Tube cap, D. Collector tube

### CAUTION

Do NOT wash your intimate zone thoroughly before urinating.  
Do NOT urinate 1 hour prior to use.  
Do NOT use if the packaging is damaged.  
For single use only.

### User instructions

- Unscrew the tube cap (C).
- Push the collector tube (D) on the Colli-Pee® funnel.
- Collect urine: The device will automatically collect the correct volume. There is no need to interrupt the urine stream. Excess urine that no longer fits into the collector tube, will flow into the toilet via the other end of the housing.
- Disconnect the collector tube from the Colli-Pee® funnel.
- Close the tube tightly. Tube cap (C) should be screwed on tightly.
- Dispose the Colli-Pee® funnel according to local regulations for plastic waste.
- Wash your hands.

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