
Colli-Pee® UCM® FV-5010


REF N00327

COLLI-PEE is a registered trademark of DNA Genotek Inc.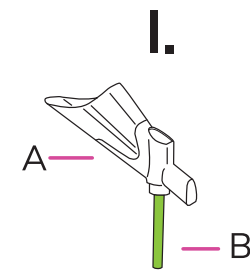

II.

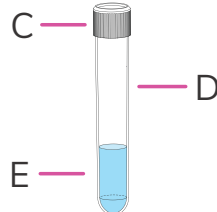Patent (www.dnagenotek.com/us/company/legal-notice.html)
© 2024 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

EC REP

Qarad EC-REP BV
Pas 257
2440 Geel
Belgium
info@qarad.com

UK RP

Qarad UK Ltd.
8 Northumberland Ave
Westminster, London WC2N 5BY
United Kingdom
info@qarad.com

DNAGENOTEK™
www.dnagenotek.com

DNA Genotek Inc.
3000-500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

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EN
FIRST-VOID URINE SELF-SAMPLING DEVICE

For standardized, volumetric collection and preservation of first-void urine samples.
Read all instructions prior to collection.

ONE PACKAGE CONTAINS

I. Colli-Pee® funnel

A. Housing, B. Floater

II. Collector tube

C. Tube cap, D. Collector tube, E. Preservative 3.4 mL UCM®

CAUTION

Do NOT wash your intimate zone thoroughly before urinating.
Do NOT urinate 1 hour prior to use.
Do NOT use if the packaging is damaged.
For single use only.
Preservative should remain inside collector tube. Do not drink the preservative. Wash with water if preservative comes into contact with skin or eyes.*

User instructions

1. Unscrew the tube cap (C).
2. Push the collector tube (D) on the Colli-Pee® funnel.
3. Collect urine: The device will automatically collect the correct volume. There is no need to interrupt the urine stream. Excess urine that no longer fits into the collector tube, will flow into the toilet via the other end of the housing.**
4. Disconnect the collector tube from the Colli-Pee® funnel.
5. Close the tube tightly. Tube cap (C) should be screwed on tightly.
6. Dispose the Colli-Pee® funnel according to local regulations for plastic waste.
7. Wash your hands.

*More information can be found on the SDS, www.dnagenotek.com
**In case of any serious incident with the device, please inform DNA Genotek and FAMHP: <https://www.famhp.be/en>

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