EN English – Instructions for sample collection	2
FR Français – Instructions de prélèvement de l'échantillon	3
ES Español – Instrucciones para la recogida de la muestra.....	4
DE Deutsch – Anweisungen zur Probenahme.....	5
NB Norsk – Instruksjoner for prøveinnsamling.....	6
SV Svenska – Anvisningar för provinsamling	7
DA Dansk – Instruktioner til indsamling af prøve	8
NL Nederlands – Instructies voor monsternamming.....	9
CS Český – Pokyny k odběru vzorku.....	10
IT Italiano – Istruzioni per la raccolta dei campioni	11
PL Polski – Instrukcje dotyczące pobierania próbek.....	12
PT Português – Instruções para a recolha de amostras.....	13
BZ Português – Brasil – Instruções para coleta de amostras	14
AR العربية – إجراء جمع عينة اللعاب.....	15
ZH 中文 – 样本采集说明	16
KO 한글 – 타액 샘플 채취용 절차	17
JA 日本語 – サンプル採取に関する指示書	18
BG Български – Инструкции за вземане на проба	19
HR Hrvatski – Upute za prikupljanje uzoraka	20
ET Eesti keel – Juhised proovivõtmiseks	21
FI Suomi – Ohjeet näytteen ottamiseen	22
EL Ελληνικά – Οδηγίες συλλογής δείγματος.....	23
HU Magyar – Mintagyűjtési utasítások.....	24
IE Gaeilge – Treoracha maidir le bailiú an sampla.....	25
LV Latviešu valoda – Norādījumi parauga paņemšanai	26
LT Lietuvių k. – Mėginio paėmimo instrukcijos.....	27
MT Malti – Istruzzjonijiet għall-ġbir ta' kampjun.....	28
RO Română – Instrucțiuni pentru colectarea probelor	29
RU Русский – Инструкции для сбора образца	30
SK Slovensky – Pokyny na odber vzorky	31
SL Slovenščina – Navodila za jemanje vzorca:	32

Collection precautions:

Ensure the sponge tip does **NOT** come into contact with any surface prior to collection.

Donor should **NOT** eat, drink, smoke, or chew gum for 30 minutes before collecting saliva sample.

Intended use: For the collection and stabilization of RNA from human saliva samples

Contents: 1 kit containing 1 mL stabilizing liquid

Warnings and precautions: Choking hazard. Caution should be used when inserting sponge into the mouth.

Wash with water if stabilizing liquid comes in contact with eyes or skin. Do **NOT** ingest. See SDS at www.dnagenotek.com

Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15°C / 25°C (59°F / 77°F)

Summary and explanation of the kit: ORAc collect™ RNA is a self-collection kit that provides the materials and instructions for collecting human saliva samples.

Label legend:





REF	Catalog number
CE	CE marking
UK	UKCA marking
MD	Medical device
DO NOT RE-USE	Do not re-use
CONSULT INSTRUCTIONS FOR USE	Consult instructions for use
CAUTION	Caution
USE BY	Use by
LOT	Lot number
EC REP	European Authorized Representative
CH REP	Swiss Authorized Representative
UDI	Unique Device Identifier
QTY	Quantity of units per package
15°C / 25°C	Storage instructions
59°F / 77°F	

**MADE IN US
FROM FOREIGN
AND DOMESTIC
MATERIALS**

Made in US from foreign and domestic materials

ENGLISH

Instructions for sample collection:

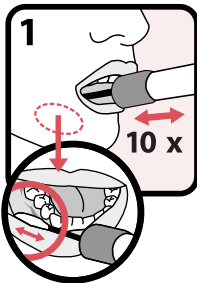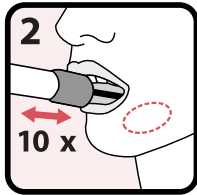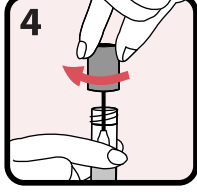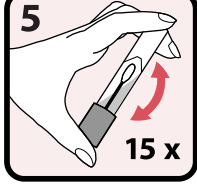	Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close-up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.
	Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.
	Hold the tube upright to prevent the stabilizing liquid inside the tube from spilling. Unscrew the cap from the collection tube without touching the sponge.
	Turn the cap upside down, insert the sponge into the tube and close cap tightly.
	Invert the capped tube and shake vigorously 15 times.

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© 2025 DNA Genotek Inc., uma subsidiária da OraSure Technologies, Inc., todos os direitos reservados.
DNA Genotek Inc.