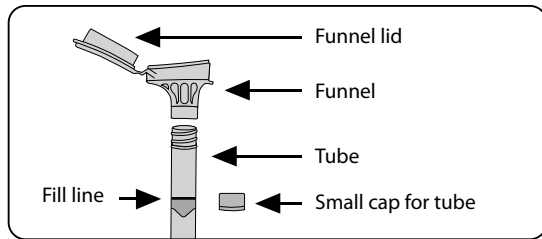
EN English – User instructions	2
FR Français – Mode d’emploi	3
ES Español – Instrucciones de uso	4
DE Deutsch – Gebrauchsanweisung	5
NB Norsk – Bruksanvisning	6
SV Svenska – Användarinstruktioner	7
DA Dansk – Brugsanvisning	8
NL Nederlands – Gebruiksaanwijzing	9
CS Český – Pokyny pro uživatele	10
IT Italiano – Istruzioni per l’utente	11
PL Polski – Instrukcja obsługi	12
PT Português – Instruções do utilizador	13
BZ Português (Brasil) – Instruções ao usuário	14
AR العربية – تعليمات المستخدم	15
ZH 中文 – 使用说明	16
KO 한글 – 사용 설명서	17
JA 日本語 – 取扱説明書	18
BG Български – Инструкции за потребителя	19
HR Hrvatski – Korisničke upute	20
ET Eesti keel – Kasutusjuhend	21
FI Suomi – Käyttöohjeet	22
EL Ελληνικά – Οδηγίες χρήστη	23
HU Magyar – Használati utasítás	24
IE Gaeilge – Treoracha úsáideora	25
LV Latviešu valoda – Lietotāja pamācība	26
LT Lietuvių k. – Naudotojo instrukcijos	27
MT Malti – Istruzzjonijiet għall-utent	28
RO Română – Instrucțiuni de utilizare	29
RU Русский – Инструкции для пользователя	30
SK Slovensky – Návod na použitie	31
SL Slovenščina – Navodila za uporabo	32

Intended use/purpose: OMNigene™•ORAL is intended for the collection and stabilization of nucleic acids from saliva.

Intended user: This product is intended for use in adults.

Contents: 1 kit containing 1 mL stabilizing liquid.

Warnings and precautions: Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at www.dnagenotek.com.

Small cap may pose a choking hazard.

Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15°C / 30°C (59°F / 86°F)

Summary and explanation of the kit:

OMNigene™•ORAL is a collection kit that provides the materials and instructions for collecting, stabilizing and maintaining during transport saliva specimens suspected of containing SARS-CoV-2 ribonucleic acid (RNA).

Collection precautions:

Do NOT eat, drink, smoke, or chew gum for 30 minutes before giving your sample.

Do NOT remove the plastic film from the funnel lid.

Label legend:


Rx ONLY	For prescription use only in the USA under EUA
REF	Catalog number
CE	CE marking
IVD	In vitro diagnostic medical device
UKCA	UKCA marking
Do not re-use	Do not re-use
Consult package insert	Consult package insert
Caution	Caution
Manufacturer	Manufacturer
EC REP	European Authorized Representative
CH REP	Swiss Authorized Representative
Collect sample by (Use by)	Collect sample by (Use by)
LOT	Lot number
UDI	Unique Device Identifier
QTY	Quantity of units per package
Storage instructions	Storage instructions
MADE IN US FROM FOREIGN AND DOMESTIC MATERIALS	Made in US from foreign and domestic materials

ENGLISH USER INSTRUCTIONS


Read all instructions prior to collection.

Procedure:

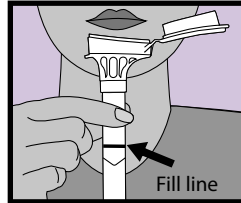
Most people take between 2 and 5 minutes to deliver a sample following steps 1 to 5.

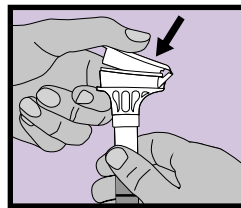
- 1 Spit into funnel until the amount of liquid (not bubbles) reaches the fill line shown in picture #1.

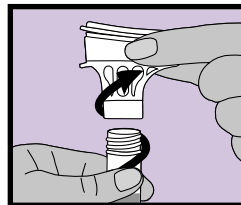
- 2 Hold the tube upright with one hand. Close the funnel lid with the other hand (as shown) by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the sample. Make sure that the lid is closed tightly.

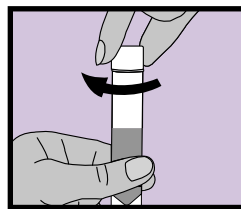- 3 Hold the tube upright. Unscrew the funnel from the tube.

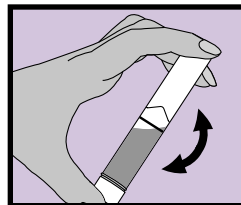- 4 Use the small cap to close the tube tightly.

- 5 Shake the capped tube for 5 seconds. Discard or recycle the funnel.

Collected specimen is potentially infectious and should be handled with appropriate biosafety practices.

Ship in accordance with applicable regulations covering transport of biological specimens.

CE UKCA Rx ONLY IVD

For In Vitro Diagnostic Use

DNAGENOTEK™

*Superior samples
Proven performance*

DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com

EC REP Qarad EC-REP BV, Pas 257, 2440 Geel, Belgium

UK Responsible Person: Qarad UK Ltd., 8 Northumberland Ave, Westminster, London, WC2N 5BY United Kingdom

CH REP Qarad Suisse S.A., World Trade Center, Avenue Gratta-Paille 2, 1018 Lausanne, Switzerland

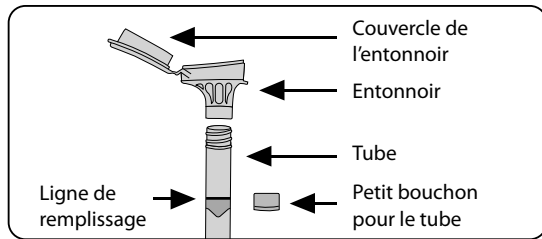
Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

OMNigene and DNA Genotek are trademarks of DNA Genotek Inc., registered under various jurisdictions. Some DNA Genotek products may not be available in all geographic regions; contact your sales representative for details. All DNA Genotek protocols, white papers and application notes are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legal/notes)

© 2025 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

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Odgovorna oseba v ZK: Qarad UK Ltd., 8 Northumberland Ave, Westminster, London, WC2N 5BY Združeno kraljestvo
CH REP Qarad Suisse S.A., World Trade Center, Avenue Gratta-Paille 2, 1018 Lausanne, Švica
© 2025 DNA Genotek Inc., podružnica OraSure Technologies, Inc., vse pravice pridržane.