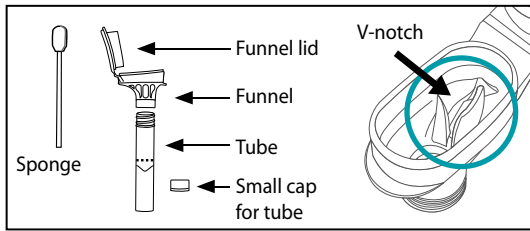
English – User instructions .....	2
Français – Mode d’emploi.....	3
Español –Instrucciones de usuario.....	4
Deutsch – Benutzeranleitung.....	5
Norsk – Bruksanvisning .....	6
Svenska – Bruksanvisning .....	7
Dansk – Brugsanvisninger .....	8
Nederlands – Gebruikersinstructies .....	9
Česky –Pokyny pro uživatele .....	10
Italiano – Istruzioni per l’uso .....	11
Polski –Instrukcje dla użytkownika .....	12
Português – Instruções do utilizador .....	13
Português – Brasil – Instruções ao usuário .....	14
العربية – تعليمات المستخدم .....	15
中文 – 用户须知 .....	17
한글 – 사용 지침 .....	18
日本語 – ユーザー指示書 .....	19
Български –Инструкции за потребителя.....	20
Hrvatski – Upute za korisnike.....	21
Eesti keel – Kasutusjuhend .....	22
Suomi – Käyttöohjeet.....	23
Ελληνικά –Οδηγίες χρήσης .....	24
Magyar –Használati utasítás .....	25
Gaeilge –treoracha úsáideora .....	26
Latviešu valoda – Lietotāja instrukcijas .....	27
Lietuvių k. –Naudotojo instrukcijos .....	28
Malti – Istruzzjonijiet tal-Utent .....	29
Română – Instrucțiuni de utilizare.....	30
Русский – Инструкция для пользователя .....	31
Slovensky – Návod na použitie.....	32
Slovenščina – Navodila za uporabnika .....	33

**Collection precautions:**

Do NOT remove the plastic film from the funnel lid.

Check sponge for damage each time before inserting into donor's mouth. Use second sponge if first sponge shows any signs of wear or tear.

Do NOT substitute with other sponges or swabs.

**Intended use:** For the assisted collection of human DNA from saliva samples

**Contents:** 1 kit containing 0.75 mL stabilizing liquid

**Warnings and precautions:**

**Choking hazards:**

- Small cap in collection kit.
- Plastic bag containing sponges.
- Caution should be used when inserting sponge into donor's mouth.

**For supervised collections:**

- Do NOT leave donor unattended.
- Do NOT allow donor to handle the sponge, small cap or packaging.
- Wash with water if stabilizing liquid comes in contact with eyes or skin. Do not ingest. See SDS at [www.dnagenotek.com](http://www.dnagenotek.com).

Report any serious incident to DNA Genotek and the competent authority in your country.

**Storage:** 15°C / 30°C (59°F / 86°F)

**Summary and explanation of the kit:**

Oragene-Dx is an assisted collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

**Label legend:**

- Flammable liquid
- REF Catalog number
- IVD In vitro diagnostic medical device\*\*
- CE CE marking
- UKCA UKCA marking
- Manufacturer
- MD Medical device\*
- Do not reuse
- Consult instructions for use
- Collect saliva by (Use by)
- LOT Lot number
- EU REP European Authorized Representative
- CH REP Swiss Authorized Representative
- Caution
- UDI Unique Device Identifier
- 15°C / 30°C Storage instructions
- 59°F / 86°F

Per applicable regulatory requirements, the device contains a swab component, which is a \*medical device, and a collection tube component, which is an \*\*in vitro device.

**User instructions**

Read all instructions prior to collection.

**Procedure:**

Ensure donor does NOT eat, drink, smoke or chew gum for 30 minutes before collecting a saliva sample.

Ensure donor is in an upright position during sample collection.

It may take up to 15 minutes to collect a saliva sample following steps 1 to 7.

	<p><b>1</b> Place one sponge in cheek pouch. Gently move the sponge along the gums and inner cheeks for 30 seconds to soak up as much saliva as possible.</p>
	<p><b>2</b> Once sponge is saturated with saliva, insert sponge in V-notch of funnel. Wring saliva out of sponge using a twisting and pushing motion against the inner wall of the V-notch. Saliva will flow into tube.</p>
	<p><b>3</b> Repeat these steps (1 to 2) USING THE SAME SPONGE until the liquid saliva (not bubbles) reaches the fill line. Check sponge for damage each time before inserting into donor's mouth. Use second sponge if first sponge shows any signs of wear or tear.  Tap tube bottom against hard surface to reduce number of bubbles.</p>
	<p><b>4</b> Hold the tube upright with one hand. Close the lid with the other hand by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva. Make sure the lid is closed tightly.</p>
	<p><b>5</b> Hold the tube upright. Unscrew the funnel from the tube.</p>
	<p><b>6</b> Use the small cap to close the tube tightly.</p>
	<p><b>7</b> Shake the capped tube for 5 seconds. Discard the funnel and sponges.</p>

See additional instructions for use PD-HB-00001 at [www.dnagenotek.com](http://www.dnagenotek.com)

Warning

For In Vitro Diagnostic Use  
For Prescription and Direct-to-Consumer Use

**DNAGENOTEK™**

Superior samples  
Proven performance

Made in Canada  
DNA Genotek Inc.  
3000 - 500 Palladium Drive  
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354  
Tel: +1.613.723.5757 • Fax: +1.613.723.5057  
[info@dnagenotek.com](mailto:info@dnagenotek.com)  
[www.dnagenotek.com](http://www.dnagenotek.com)

Novosanis NV, Bijkhoevelaan 32C, 2110 Wijnegem, Belgium Email: [EUAR@novosanis.com](mailto:EUAR@novosanis.com)

UK Responsible Person: Emergo Consulting (UK) Limited c/o Cr360 – UL International, Compass House, Vision Park Histon, Cambridge, CB24 9BZ

Arazy Group Swiss GmbH, Bruderholzallee 53, 4059 Basel, Schweiz Email: [swiss.ar@arazygroup.com](mailto:swiss.ar@arazygroup.com)

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

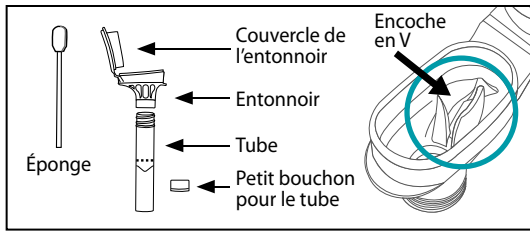
Oragene and DNA Genotek are trademarks of DNA Genotek Inc.

Some DNA Genotek products may not be available in all geographic regions; contact your sales representative for details. All DNA Genotek protocols, white papers and application notes are available in the support section of our website at [www.dnagenotek.com](http://www.dnagenotek.com).

Patent ([www.dnagenotek.com/legal/notes](http://www.dnagenotek.com/legal/notes))

© 2024 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

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© 2024 DNA Genotek Inc., dcérska spoločnosť OraSure Technologies, Inc., všetky práva vyhradené.