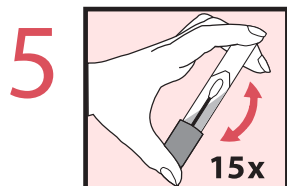
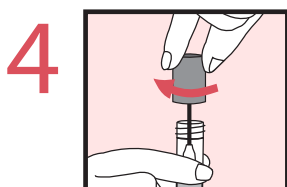
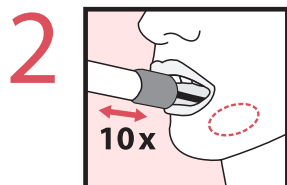
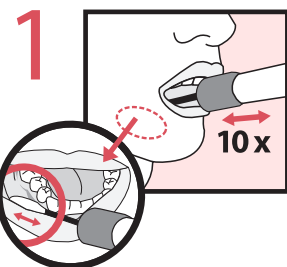
For IVD, For Prescription Use Only and For Use Under Emergency Use Authorization.

DNAGENOTEK™

ORE-100
DNA Genetek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Tel: +1.613.723.5757
info@dnagenotek.com
www.dnagenotek.com

Patent (www.dnagenotek.com/legalnotices)
PD-PR-00772 Issue 6/2023-12

EN ENGLISH
User instructions

Read all instructions prior to collection.

Collection precautions:
Ensure the sponge tip does NOT come into contact with any surface prior to collection.

Donor should NOT eat, drink, smoke, or chew gum for 30 minutes before collecting saliva sample.

Intended use: For the collection and stabilization of RNA from human saliva samples

Contents: 1 kit containing 1 mL stabilizing liquid

Warnings and precautions:
Choking hazard. Caution should be used when inserting sponge into the mouth.

Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at www.dnagenotek.com

Report any serious incident to DNA Genetek and the competent authority in your country.

Storage: 15°C | 25°C (59°F | 77°F)

Summary and explanation of the kit:
ORACollect-RNA is a self-collection kit that provides the materials and instructions for collecting human saliva samples.

Instructions for sample collection:

1 Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close-up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.

2 Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.

3 Hold the tube upright to prevent the stabilizing liquid inside the tube from spilling. Unscrew the cap from the collection tube without touching the sponge.

4 Turn the cap upside down, insert the sponge into the tube and close cap tightly.

5 Invert the capped tube and shake vigorously 15 times.

Label legend:

	Catalog number
	For Prescription Use Only
	In vitro diagnostic medical device**
	CE marking
	UKCA marking
	Manufacturer
	Medical device*
	Do not reuse
	Consult instructions for use
	Use by
	Lot number
	European Authorized Representative
	Swiss Authorized Representative
	Unique Device Identifier
	Storage instructions

Per applicable regulatory requirements, the device contains a swab component, which is a *medical device, and a collection tube component, which is an **in vitro device.

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OraSure Technologies, Inc., all rights reserved.